**Dental Etching Gel • For Acid Etching Dentin, Enamel and Glass Ionomer Cement**

**38% PHOSPHORIC ACID**

Etch-Rite is a thixotropic phosphoric acid etch gel that stays in place and will not run onto exposed tissue. Syringe dispensing through pre-bent small gauge applicator tips offers precise placement of gel even in relatively inaccessible areas. Etch-Rite washes off easily with water.

**PRODUCT DESCRIPTION**

38% phosphoric acid dental etching gel

**INDICATIONS**

For acid etching dentin, enamel and glass ionomer cement

**CONTRAINDICATIONS**

Do not place on soft tissues

**DIRECTIONS FOR USE**

1. Prepare tooth and cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophylactic paste. Rinse and dry thoroughly.
2. Securely attach 25-gauge luer lock applicator tip to syringe. Start flow on pad; not in patient's mouth. Do not push on plunger with thumb. For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
3. Recommended etching time for enamel is 15-20 seconds and for dentin is 15 seconds. To the extent possible, apply Etch-Rite first to enamel and then to dentin. Leave on dentin for 15 seconds. Cap syringe immediately after use.
4. Rinse off Etch-Rite with copious amounts of water. Do not disturb this surface.
5. Leave surface wet, or dry surface with clean, uncontaminated

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air, depending on the instructions for use for the bonding system you are using.

**Note:** Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

**WARNING:**

If Etch-Rite contacts skin or soft tissue, rinse off immediately with copious amounts of water.

**STORAGE AND HANDLING**

• Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition. • Shelf life of unopened product: 2 years from date of manufacture. • Re-cap immediately after use.

**REPORTING OF SERIOUS INCIDENTS**

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to: Pulpdent Corporation  
80 Oakland Street, Watertown, MA 02472 USA  
Tel: 617-926-6666 / Fax: 617-926-6262  
email: pulpdent@pulpdent.com

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>.

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

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**Notification des incidents graves**

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#### Ravelli S.p.A.
#### Hofmeester Dental B.V.
#### Gama Dental AB
#### Lifco Dental AB