Gebrauchsinformation**DE**

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Polydimethylsiloxane

Indications

- Temporary cementation of provisional restorations
- Temporary cementation of provisional restorations on implant abutments
- Temporary cementation of final crowns and bridges
- Temporary cementation of final crowns and bridges on implant abutments

Clinical Time

Automix

Working Time

≤ 0:50 min

Setting Time

≥ 2:00 min

Contra-Indications

Wenn used as directed, there are no known contra-indications.

Side Effects / Interactions

A-Silicones have a high degree of biological compatibility with no known harmful reactions or secondary effects on patients or

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Application

1. Gently dry the prepared tooth surface by using oil-free compressed air. Do not overdry. The best results will be achieved, when the prepared tooth surface looks moist (shiny).
2. Cleanse and dry the inside of the restoration.
3. Remove the syringe plug on the syringe and discard.
4. Extrude a small amount of material directly out of the syringe onto a paper towel/tissue until it is evident that base and catalyst are being ejected. This ensures proper mixing.
5. Immediately wipe the syringe orifice to remove excess material.
6. Attach the Mixing Tip to the syringe by aligning the prongs inside the Mixing Tip with the syringe orifice. Seat onto the syringe and twist clockwise (90°) to lock in place.
7. Apply TempoSIL 2 directly from the syringe into the provisional. Seal the provisional, applying slight pressure. Secure undisturbed for 30–60 s.
8. After setting, remove the excess material (e.g. scalar).
9. After syringe use, disinfect used Mixing Tip and store the syringe with the used Mixing Tip attached.

Mixing Tip

Always store the TempoSIL 2 syringes with the Mixing Tip attached. This seals the syringe and prevents contamination. Remove used Mixing Tip prior to using the syringe again. Bleed syringe to check that material ejects evenly. Attach new Mixing Tip.

Important

Skin secretions, latex gloves and surfaces contaminated by latex glove may interfere with the setting process of A-Silicones. Touch material and the preparation with thoroughly washed and rinsed hands (wash for 15 s with a detergent, rinse with water for another 15 s) or use latex-free gloves. Compounds containing eugenol, hemostatic agents or some anaesthetics may also impede setting. High temperatures accelerate and low temperatures decelerate the setting process.

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