



ULTRADENT
PRODUCTS, INC.

Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**



UMBRELLA™

TONGUE, LIP AND CHEEK RETRACTOR

Umbrella Tongue, Lip, & Cheek Retractor

Description

Core Messages

- Maximizes patients' comfort
- Does not pull or stretch cheeks or lips
- Does not initiate gag reflex for most patients
- Innovative tongue retraction
- Easy handling and placement
- Opens and maintains space between teeth, lips, and cheeks
 - at the same time it keeps saliva away from dentition
 - this improves access for HVE

Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**



FEATURES & BENEFITS

Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

One solution for various indications, including

- Anterior restorations/crowns
- Bi-lateral posterior restorative
- Cleanings
- Sealants
- Ortho bracket bonding
- In-office tooth whitening
- Impressions
- Scanning
- Inlays/onlays
- Veneers
- Implants
- Gaggers
- Anxious patients
- Difficult-to-manage tongues
- Long procedures
- etc.

Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Dentists love Umbrella for ...

- Ideal lip & cheek retraction
 - clear bilateral access to both arches without any need to reposition
 - open working space at any area of the mouth, free of tongue
- Innovative tongue retraction
 - keeps tongue out of the way without causing discomfort
 - tongue comfortably rests behind tongue guard



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Dentists love Umbrella for ...

- Channeling of the saliva & fluid control
 - opens space between teeth, lips and cheeks in a way that encourages saliva to accumulate away from the dentition
 - optimal access for HVE



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Dentists love Umbrella for ...

- Anatomically shaped bumpers
 - clinician can rest hand on patient's mouth without causing any pain or discomfort to the patient
- Reliability
 - securely stays in place, e.g. if bite checks are needed



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Dentists love Umbrella for ...

- Easy handling & placement
 - due to soft and flexible material
- Disposable single-use device
 - perfect cross-infection control
 - no sterilization needed



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Patients love Umbrella for ...

- Maximized patient comfort
 - retraction without discomfort or pain
 - device's geometry fits the anatomy of the mouth
 - no need for assistants/dentists to keep cheeks away
 - no more tense jaws, no more stretched and cracked lips



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Patients love Umbrella for ...

- Flexible, spring-loaded design
 - naturally and gently keeps mouth open
 - no effort for patient
 - relaxed jaws
- Improved fluid control
 - minimized pooling



Umbrella Tongue, Lip, & Cheek Retractor

Features & Benefits

Patients love Umbrella for ...

- Relief effect to “gaggers”
 - no initiation of the gag reflex for most patients
- Especially suitable for anxious patients
 - convincing comfort concept
 - improved overall experience
- Faster procedures



Umbrella Tongue, Lip, & Cheek Retractor


Features & Benefits

Anatomically positioned bumpers

- clinician can rest hand on patient's mouth
- no pain points for the patient

Innovative tongue guard

- ideal retraction
- rests comfortably under the tongue

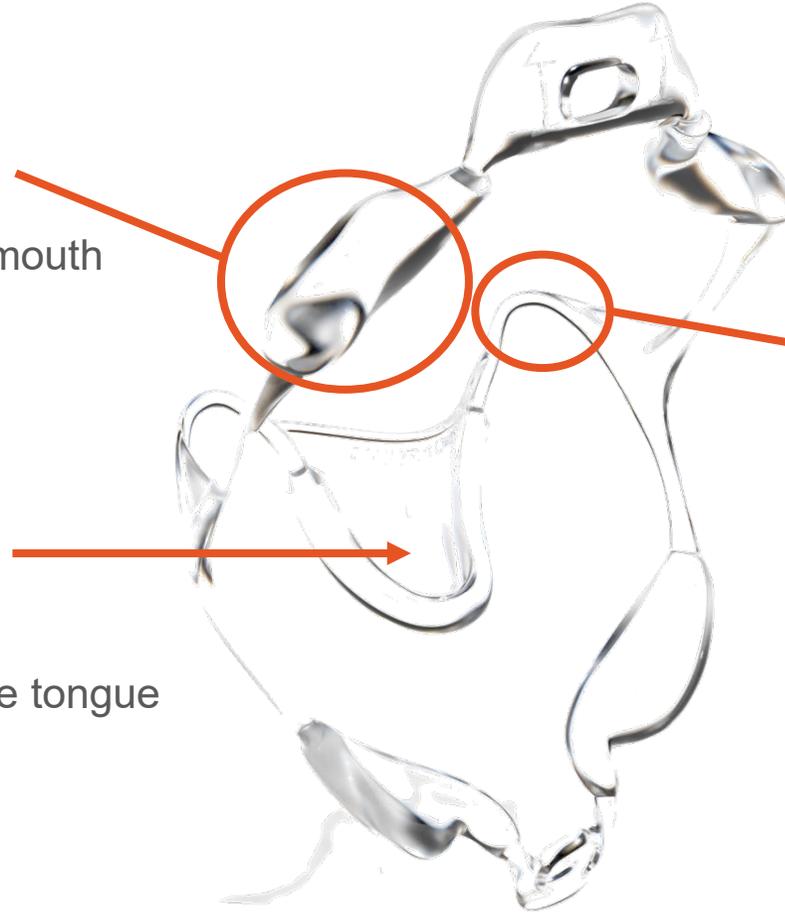

Naturally spring-loaded

- patients keep mouth open without effort
- securing maximal patient comfort
- full mouth closure is still possible for occlusion checks

Umbrella Tongue, Lip, & Cheek Retractor

Summary

- Creates and maintains clear access to the working environment
- Encourages saliva to accumulate away from dentition
- Does not initiate gag reflex
- Does not cause pain or discomfort
 - One device for various procedures
 - Combines multiple advantages in one easy-to-use product
 - Ensures a better overall experience for dentist and patient

Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**



HOW-TO

Umbrella Tongue, Lip, & Cheek Retractor Handling

STEP 1.



Simply press the tabs on the Umbrella™ soft tissue retractor together to prepare it for insertion.

STEP 2.



Ask the patient to place the tip of their tongue on the roof of their mouth.

STEP 3.



Choose one side of the mouth in which to start, and then comfortably insert the other side of the retractor into the cheek.

STEP 4.



Use the tabs to center the retractor with the patient's mouth.

STEP 5.



Check that the patient's tongue is resting comfortably behind the guard, ensuring easy access.

Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**



PACKAGING & PRICING

Umbrella Tongue, Lip, & Cheek Retractor

Packaging & Pricing



Medium size fits
most adult patients

Available Kits & Pricing

SKU	Description	Dealer Price	Recommended Retail Price
4870	Umbrella Cheek Retractor Medium 5pk (Clear) <i>5x Umbrella Cheek Retractor</i>	5,19 EUR	8,65 EUR
4871	Umbrella Cheek Retractor Medium 20pk (Clear) <i>20x Umbrella Cheek Retractor</i>	20,46 EUR	34,10 EUR
5162	Umbrella Cheek Retractor Medium 40pk (Clear) <i>40x Umbrella Cheek Retractor</i>	40,74 EUR	67,90 EUR



Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**



FURTHER INFORMATION

Umbrella Tongue, Lip, & Cheek Retractor

Further Information

Videos

- How to use Umbrella
 - [YouTube](#)
- Umbrella Procedures
 - [YouTube](#)



Beautiful Access,
Practical Isolation, in a
**Comfortable
Way!**

